

# Masimo SafetyNet Alert®

Oxygen Monitoring & Alert System






© 2023 Masimo. All rights reserved. 302386/11129G-0523

# SYSTEM COMPONENTS



### Masimo Chip

- Reusable chip snaps into the sensor and communicates with the Home Medical Hub.

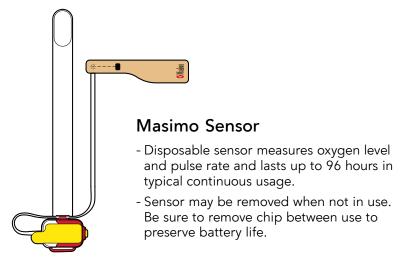




## Masimo Halo App

- Displays oxygen, pulse rate and other data.
- Cloud sends alerts to Home Medical Hub and mobile device.
- Compatible with most mobile devices. (Mobile device not included.)



#### Home Medical Hub

- Collects data from Masimo Sensor.
- Sends data to smartphone app.
- Store the chip in the holder on top of Home Medical Hub.

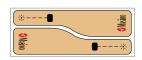

## Replacement Tapes


- Additional tapes for replacement as needed.

#### SET UP APP & CONNECT HOME MEDICAL HUB

1. Download the app and create an account.

Masimo HALO

Connect Home Medical Hub to power.

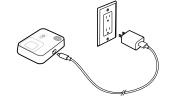

3. Pair Home Medical Hub with phone (Smartphone Bluetooth should be ON).




4. Connect Home Medical Hub to Wifi\*. Follow step-by-step instructions on app to set up system.



\*Only compatible with 2.4 GHz wireless networks.

#### SET UP SENSOR

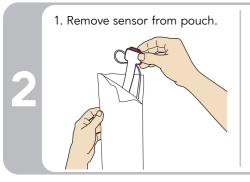

2. Peel off yellow label to activate battery.



3. Place sensor on wrist as shown and carefully thread the tip of the strap through the open red loop.



4. Wrap the loose end of the strap around your wrist and press to



#### APPLY SENSOR




2. Wrap tape around finger so black square ( ■ ) is on finger pad directly across from star (※).



3. Squeeze top and bottom of finger to secure tape.




4. Remove plastic film and continue to wrap tape around finger.

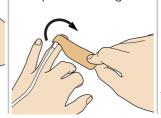


5. Adjust cable to comfortable length.




#### PAIR AND INSERT CHIP

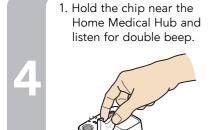

2. Insert the chip into open compartment on the sensor.

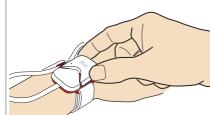

3. Press down on the chip to secure.

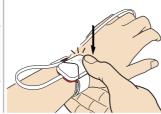

Click







Look for light to confirm chip connection to sensor

### CONFIRM SUCCESSFUL PAIRING


 Once the chip is inserted, the Home Medical Hub Bluetooth Status LED will turn off and the Status LED will turn green. The chip light indicator will turn solid blue.





Visit http://www.mymasimo.co.uk/safetynet-alert to see full list of supported devices, purchase additional sensors and view other resources, including videos and instructions.

#### CHANGING THE REPLACEMENT TAPE

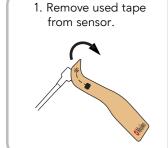

2. Attach new tape in same direction as used tape.



3. Attach sensor to finger.



4. Peel the remaining tape

